


## External Storage of Patient Exams

Images and Clips can be reloaded for each patient and a specific exam can be reviewed. Specific measurements can be taken and saved on the reloaded images.

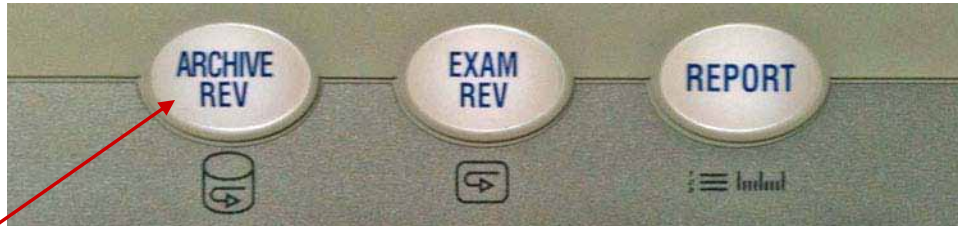

1. Press the Archive Rev button.

2. Press the set key to highlight the patient's exam.
3. Next, move the cursor to the last name that you want to export and let it hover over it. Press Shift and Set buttons to highlight all names between the two exams.
4. Move cursor with track ball to the Export menu and press set or use Press the software button that corresponds to the export monitor menu.



5. Select the export method of choice in the drop down menu.
6. Move the cursor to "OK" and press the set key.

